



DATA SHEET

Laboratory and Courier Supplies

ACCESSORIES



Gel Packs To keep specimens at the correct temperature use Duramark Gel Ice Packs. Three temperature ranges are available +30°F, +10°F, and -10°F. They are designed to fit into our lock boxes, car caddies, courier totes, and transport containers.

GEL ICE Packs

+30°F “F” formula

ICE 8

72 per case

ICE 12

42 per case

ICE 24

24 per case

+10°F “X” formula

ICE 8X

72 per case

ICE 12X

42 per case

ICE 24X

24 per case

-10°F “Y” formula

ICE 8Y

72 per case

ICE 12Y

42 per case

ICE 24Y

24 per case

Standard freezer temperature is -20° C or -5° F. This will freeze either the standard gel pack “F” formula (+30° F) or the “X” formula (+10° F).

Cryo-freezers (shock freezers) have a temperature of around -20° C to -60° C. This will completely freeze the “Y” formula gel ice. The “Y” formula is useful in conjunction with Dry Ice. If used in this method, the “Y” formula can be stored in a standard cooler. While this will not freeze the pack completely, it will when used with Dry Ice. This will help cut down on the Dry Ice costs while also maintaining a longer cold duration for the shipment. No tests have been conducted to determine exactly how much longer utilizing the “Y” formula extends the coolant time of any particular shipment.

Specimen Integrity & Public Safety

Duramark, Inc. 11418 Knott Street, Garden Grove, CA 92841

☎ 714.828.2800 ☎ 714.828.1722 www.duramark.com



Made in
USA



DATA SHEET

Laboratory and Courier Supplies

ACCESSORIES

Absorbent or Desiccant Pads

Spills happen that is why DuraMark has both absorbent and desiccant pads available. They are designed to work with our courier totes and shipping containers. Our pads offer superior absorbent rates.

Absorbent Pads

Model Number: 8L816

Capacity: 240 gsm

Size: 8" x 16.5"

Desiccant Pads

Model Number: DL-36

Capacity: 54-164 ml

Size: 3" x 6"

Model Number: DL-56

Capacity: 90 – 270 ml

Size: 5" x 6"

BioHazard Spill Kits

A specimen is dropped, what do you do? The simple solution is to grab a DuraMark Duratech spill kit. It has virtually everything that you need to deal with a specimen spill. Every courier needs one in his car and specimen tote bag. Every lab needs a few within handy reach. Call today for yours.

The kit includes one (1) of each of the following:

Pair of gloves, gown, mask with eye shield, shoe covers, pair of disposable waste scoops, antimicrobial hand wipe, packet body fluid treatment and encapsulant, surface disinfectant wipe, black waste bag, red biohazard bag.

Model Number: Duratech Splkt

Size: 12.5" x 8" x 2"



Specimen Integrity & Public Safety

Duramark, Inc. 11418 Knott Street, Garden Grove, CA 92841

☎ 714.828.2800 📠 714.828.1722 www.duramark.com

