



# Laboratory and Courier Supplies



**ConstanTemp®**

## The patented ConstanTemp® maintains Room, Refrigerated and Frozen Specimen Integrity

This small handy fabric ConstanTemp® was initially designed to maintain clinical study specimen integrity for one of the largest laboratories in the USA. This product is capable of maintaining specimens at room, refrigerated and frozen temperatures.

For frozen specimens, simply put the ConstanTemp® in the freezer over night or until it reaches -4°F (-20°C). When the specimen is frozen, just insert it into the handy pouch. It will stay frozen for over 19-hours at an ambient temperature of 72°F.

The ConstanTemp® is just as easy to use for refrigerated specimens. Simply put the pouch into the refrigerator over night or until it reaches 35°F (2°C) and it is ready to use. It will maintain 35-46°F (2-8°C) in either a hot or cold environment for hours.

For room temperature specimens, simply leave the ConstanTemp® on the counter or shelf in a 68-77°F (20-25°C) room and place the specimen inside. It will maintain this temperature for hours.



**Interior View**

### **ConstanTemp® Specifications:**

The ConstanTemp® has 6 tube pouches. Included with the ConstanTemp® are 2 specially formulated gel packs. A rear vinyl pocket is provided for temperature identification our pouch labeling.

**Size:** 8" high x 6" wide x 2" deep with heavy-duty zipper.

**Media:** Special gel formula (+10° F) that holds its temperature for extend periods.

**Insulation:** ¼" Volara

**Generic Printing:** Biohazard Symbol

**Model Number:** CTT-1000

**Customized Design:** Special colors, printing and sizes available